Instructions for Use

VZV IgM ELISA

IVD

CE

REF EIA-3524

Σ 96

DRG Instruments GmbH, Germany
Frauenbergstraße. 18, D-35039 Marburg
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50
Website: www.drg-diagnostics.de
E-mail: drg@drg-diagnostics.de

DRG International, Inc., USA
841 Mountain Ave., Springfield, NJ 07081
Phone: (973) 564-7555, Fax: (973) 564-7556
Website: www.drg-international.com
E-mail: corp@drg-international.com

Please use only the valid version of the package insert provided with the kit.
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Arbeitsanleitung.
Si prega di usare la versione valida dell'inserto del pacco a disposizione con il kit.

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1 INTRODUCTION

1.1 Intended Use

The **DRG VZV IgM Enzyme Immunoassay Kit** provides materials for the **qualitative** and **semiquantitative** determination of IgM-class antibodies to Varicella zoster Virus in human serum or plasma (EDTA, heparin or citrate plasma).

This assay is intended for **in vitro diagnostic use only**.

1.2 Summary and Explanation

Varicella-Zoster Virus (human herpes virus 3, HHV-3) belongs to the a-subfamily of herpesviridae. The virus particles measure about 145 nm in diameter. They consist of double stranded DNA, are surrounded by an icosahedral protein capsid and an envelope which contains both host cells and viral components. The virus is usually transmitted in respiratory secretions, and a single serotype causes varicella (Chickenpox), a highly infectious childhood disease, and zoster (shingles), a neurodermic disease; both diseases are found worldwide. Varicella is the acute disease which follows primary contact with the virus, whereas zoster is the response of the partially immune host to a reactivation of the varicella virus present in the body in latent form. Varicella is endemic, most commonly affected are children between 2 and 6 years of age. The course of disease is usually mild and complicated only in immunocompromised children. Rare fatal cases show multiple necrotic lesions in brain, lung (varicella pneumonia), kidneys (hemorrhagic nephritis), spleen, bone marrow, and occasionally in the intestinal tract. The lethality of varicella is below 0.1%. In the infrequent adult infections the disease is more severe, and complications are to be expected in about 5% of all cases. Zoster is of low incidence and appears with increasing frequency and severity with advancing age. Usually the process remains localized, generalization is frequently encountered in a state of immunosuppression. Fatal cases are very rare and nearly always caused by an underlying disease.

2 PRINCIPLE OF THE TEST

The **DRG VZV IgM ELISA** Kit is a solid phase enzyme-linked immunosorbent assay (ELISA)

Patient samples are diluted with *Sample Diluent* and additionally incubated with *IgG-RF-Sorbent*, containing hyper-immune anti-human IgG-class antibody to eliminate competitive inhibition from specific IgG and to remove rheumatoid factors. This pretreatment avoids false negative or false positive results.

Microtiter wells as a solid phase are coated with inactivated Varicella zoster Virus antigen (wild strain isolate).

Diluted patient specimens and ready-for-use controls are pipetted into these wells. During incubation Varicella zoster Virus-specific antibodies of positive specimens and controls are bound to the immobilized antigens.

After a washing step to remove unbound sample and control material horseradish peroxidase conjugated anti-human IgM antibodies are dispensed into the wells. During a second incubation this anti-IgM conjugate binds specifically to IgM antibodies resulting in the formation of enzyme-linked immune complexes.

After a second washing step to remove unbound conjugate the immune complexes formed (in case of positive results) are detected by incubation with TMB substrate and development of a blue color. The blue color turns into yellow by stopping the enzymatic indicator reaction with sulfuric acid.

The intensity of this color is directly proportional to the amount of Varicella zoster Virus-specific IgM antibody in the patient specimen. Absorbance at 450 nm is read using an ELISA microtiter plate reader.

3 WARNINGS AND PRECAUTIONS

- This kit is for in vitro diagnostic use only. For professional use only.
- Before starting the assay, read the instructions completely and carefully. Use the valid version of the package insert provided with the kit. Be sure that everything is understood.
- All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. All reagents, however, should be treated as potential biohazards in use and for disposal.
- Avoid contact with Stop Solution containing 0.2 mol/L H₂SO₄. It may cause skin irritation and burns.
- TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant volume of water and skin with soap and abundant water. Wash contaminated objects before reusing them. If inhaled, take the person to open air.
- The microplate contains snap-off strips. Unused wells must be stored at 2 °C to 8 °C in the sealed foil pouch and used in the frame provided
- Pipetting of samples and reagents must be done as quickly as possible and in the same sequence for each step.
- Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution colored. Do not pour reagents back into vials as reagent contamination may occur.
- Mix the contents of the microplate wells thoroughly to ensure good test results. Do not reuse microwells.
- Do not let wells dry during assay; add reagents immediately after completing the rinsing steps.
- Allow the reagents to reach room temperature (21 °C to 26 °C) before starting the test. Temperature will affect the absorbance readings of the assay. However, values for the patient samples will not be affected.
- Never pipette by mouth and avoid contact of reagents and specimens with skin and mucous membranes.
- Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.
- Wear disposable latex gloves when handling specimens and reagents. Microbial contamination of reagents or specimens may give false results.
- Handling should be in accordance with the procedures defined by an appropriate national biohazard safety guideline or regulation.
- Do not use reagents beyond expiry date as shown on the kit labels.
- All indicated volumes have to be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiter plate readers.
- Do not mix or use components from kits with different lot numbers. It is advised not to exchange wells of different plates even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the plates may result slightly different.
- Chemicals and prepared or used reagents have to be treated as hazardous waste according the national biohazard safety guideline or regulation.
- For information on hazardous substances included in the kit please refer to Safety Data Sheets.
Safety Data Sheets for this product are available upon request directly from DRG.

4 REAGENTS

4.1 Reagents provided

Symbol	Quantity	Description	Preparation
Microtiterwells	12 x 8 wells (break apart)	Microtiter plate Wells coated with Varizella Zoster-Virus antigen.	Ready to use
Sample Diluent	1 x 100 mL	Sample Diluent * colored yellow, pH 7.2 ± 0.2	Ready to use
IgG-RF-Sorbent	1 x 6,5 mL	IgG-RF-Sorbent * colored yellow; Contains anti-human IgG-class antibody.	Ready to use
Neg. Control	1 x 2 mL	Negative Control * Colored yellow, yellow cap.	Ready to use
Cut-off Control	1 x 2 mL	Cut-off Control * Colored yellow, black cap.	Ready to use
Pos. Control	1 x 1 mL	Positive Control * Colored yellow, red cap.	Ready to use
Enzyme Conjugate	1 x 20 mL	Enzyme Conjugate * Colored red, antibody to human IgM conjugated to horseradish peroxidase.	Ready to use
Substrate Solution	1 x 14 mL	Substrate Solution Contains 3,3',5,5'-tetramethylbenzidine (TMB). <i>Keep away from direct sun light.</i>	Ready to use
Stop Solution	1 x 14 mL	Stop Solution Contains 2 % H ₂ SO ₄ . <i>Avoid contact with the stop solution. It may cause skin irritations and burns.</i>	Ready to use
Wash Solution	1 x 30 mL	Wash Solution, 20X concentrate *	See "Reagent Preparation".
	1 x	Cover foil	
	1 x	Instructions for Use	
	1 x	Certificate of Analysis (CoA)	
		* Contain(s) non-mercury preservative.	

4.1.1 Material required but not provided

- A microtiter plate calibrated reader (450/620 nm ±10 nm) (e.g. the DRG Instruments Microtiter Plate Reader)
- Calibrated variable precision micropipettes
- Incubator 37 °C
- Manual or automatic equipment for rinsing wells
- Vortex tube mixer
- Deionised or (freshly) distilled water
- Timer
- Absorbent paper

4.2 Storage Conditions and stability of the Kit

When stored at 2 °C to 8 °C unopened reagents will retain reactivity until expiration date. Do not use reagents beyond this date. Opened reagents must be stored at 2 °C to 8 °C. Microtiter wells must be stored at 2 °C to 8 °C. Once the foil bag has been opened, care should be taken to close it tightly again.

Opened kits retain activity for two months if stored as described above.

4.3 Reagent Preparation

Allow all reagents and required number of strips to reach room temperature prior to use.

Wash Solution

Add fresh and germ-free distilled water to the 20X concentrated Wash Solution.

Dilute the complete content of the vial **1 + 19** (30 mL *Wash Solution* + 570 mL distilled water) to a final volume of 600 mL. If crystals have formed in the wash solution concentrate, ensure that they are completely transferred and dissolved into the solution.

This diluted wash solution must have a pH value of 7.2 ± 0.2 .

Stability after dilution:	at 20 °C to 25 °C	1 week
---------------------------	-------------------	--------

4.4 Disposal of the Kit

The disposal of the kit must be made according to the national regulations. Special information for this product is given in the Material Safety Data Sheets.

4.5 Damaged Test Kits

In case of any severe damage to the test kit or components, DRG has to be informed in writing, at the latest, one week after receiving the kit. Severely damaged single components should not be used for a test run. They have to be stored until a final solution has been found. After this, they should be disposed according to the official regulations.

5 SPECIMEN

Serum or plasma (EDTA, lithium heparin or citrate plasma) can be used in this assay.

Samples containing sodium azide should not be used in the assay.

In general, it should be avoided to use hemolytic, icteric or lipaemic samples. For further information refer to chapter Matrix Interference.

5.1 Specimen Collection

Serum:

Collect blood by venipuncture (e.g. Sarstedt Monovette for serum), allow to clot, and separate serum by centrifugation at room temperature. Do not centrifuge before complete clotting has occurred. Patients receiving anticoagulant therapy may require increased clotting time.

Plasma:

Whole blood should be collected into centrifuge tubes containing anti-coagulant (e.g. Sarstedt Monovette with the appropriate plasma preparation) and centrifuged immediately after collection.

5.2 Specimen Storage and Preparation

Specimen must be stored tightly capped prior to performing the assay. If stored frozen, freeze only once. Thawed specimens must be inverted several times prior to testing.

Stability	at 2 °C to 8 °C	5 days
	at -20 °C (in aliquots)	up to 12 months

5.3 Specimen Dilution

Prior to assaying each patient specimen is first to be diluted with *Sample Diluent*. For the absorption of rheumatoid factor these prediluted samples then have to be incubated with *IgG-RF-Sorbent*

1. Dilute each patient specimen **1+50** with *Sample Diluent*,
e.g. 10 µL of specimen + 0.5 mL of *Sample Diluent*. **Mix well**.
2. Mix well the *IgG-RF-Sorbent* before use.
3. Dilute this prediluted sample **1+1** with *IgG-RF-Sorbent*
e.g. 60 µL prediluted sample + 60 µL *IgG-RF-Sorbent*. **Mix well**
4. **Let stand at room temperature for at least 15 minutes, up to a maximum of 2 hours and mix well again.**
5. Take 100 µL of these pretreated samples for the ELISA.

Please note: Controls are ready for use and must not be diluted!

Take 100 µL of the pretreated sample for the ELISA. Use only fresh pretreated samples!

6 ASSAY PROCEDURE

6.1 General Remarks

- **It is very important to bring all reagents, samples and controls to room temperature before starting the test run!**
- Once the test has been started, all steps should be completed without interruption.
- Use new disposal plastic pipette tips for each standard, control or sample in order to avoid cross contamination
- Absorbance is a function of the incubation time and temperature. Before starting the assay, it is recommended that all reagents are ready, caps removed, all needed wells secured in holder, etc. This will ensure equal elapsed time for each pipetting step without interruption.
- As a general rule the enzymatic reaction is linearly proportional to time and temperature.
- Close reagent vials tightly immediately after use to avoid evaporation and microbial contamination.
- To avoid cross-contamination and falsely elevated results pipette patient samples and dispense conjugate without splashing accurately to the bottom of wells.
- During 37 °C incubation cover microtiter strips with foil to avoid evaporation.

6.2 Assay Procedure

Prior to commencing the assay, dilute *Wash Solution*, **prepare patient samples as described in point 5.3** and establish carefully the **distribution and identification plan** supplied in the kit for all specimens and controls.

1. Select the required number of microtiter strips or wells and insert them into the holder.

Please allocate at least:

1 well (e.g. A1) for the *Neg. Control*,
2 wells (e.g. B1+C1) for the *Cut-off Control* and
1 well (e.g. D1) for the *Pos. Control*.

It is left to the user to determine controls and patient samples in duplicate.

2. Dispense

100 µL of *Neg. Control* into well A1
100 µL of *Cut-off Control* into wells B1 and C1
100 µL of *Pos. Control* into well D1 and
100 µL of each pre treated sample with new disposable tips into appropriate wells.

3. Cover wells with foil supplied in the kit. Incubate for **60 minutes at 37 °C**.

4. Briskly shake out the contents of the wells.

Rinse the wells **5 times** with diluted *Wash Solution* (**300 µL per well**). Strike the wells sharply on absorbent paper to remove residual droplets.

Important note:

The sensitivity and precision of this assay is markedly influenced by the correct performance of the washing procedure!

5. Dispense **100 µL Enzyme Conjugate** into each well.

6. Incubate for **30 minutes at room temperature (20 °C to 25 °C)**.

Do not expose to direct sun light!

7. Briskly shake out the contents of the wells.

Rinse the wells **5 times** with diluted *Wash Solution* (300 µL per well). Strike the wells sharply on absorbent paper to remove residual droplets.

8. Add **100 µL of Substrate Solution** into all wells.

9. Incubate for **exactly 15 minutes at room temperature (20 °C to 25 °C) in the dark**.

10. Stop the enzymatic reaction by adding **100 µL of Stop Solution** to each well.

Any blue color developed during the incubation turns into yellow.

Note: Highly positive patient samples can cause dark precipitates of the chromogen!

11. Read the optical density at **450/620 nm** with a microtiter plate reader **within 30 minutes** after adding the *Stop Solution*.

6.3 Measurement

Measure the absorbance of all wells **at 450 nm** and record the absorbance values for each control and patient sample in the distribution and identification plan.

Dual wavelength reading using 620 nm as reference wavelength is recommended.

Where applicable **calculate the mean absorbance values** of all duplicates.

7 RESULTS

7.1 Validation of the Test Run

The test run may be considered valid provided the following criteria are met:

Neg. Control in A1: Absorbance value **lower than 0.200**

Cut-off Control in B1/C1: Absorbance value **between 0.350 – 0.850**

Pos. Control in D1: Absorbance value **between 0.650 – 3.000**

7.2 Calculation and Interpretation of Results

7.2.1 Semiquantitative method

Where applicable **calculate the mean OD values** of all duplicates.

Calculation of Cut-off [CO]

Calculate the mean OD value of the duplicate determination of the Cut-off Control.

$$\text{Example: } \frac{0.440 + 0.460}{2} = 0.450 = \text{CO}$$

7.2.2 Calculation of Semiquantitative Results in DRG Units [DU]

$$\frac{\text{Mean OD}_{\text{sample}} \times 10}{\text{CO}} = \text{DRG Units [DU]}$$

$$\text{Example: } \frac{1.580 \times 10 \text{ DU}}{0.450} = 35 \text{ DU}$$

7.2.3 Interpretation of Semiquantitative Results

Cut-off value: 10 DU	NEGATIVE:	< 9 DU
	GREY ZONE (equivocal):	9 – 11 DU
	POSITIVE:	> 11 DU

7.2.4 Interpretation of Qualitative Results

NEGATIVE Patient sample (mean) OD value more than 10 % below CO
(Mean $\text{OD}_{\text{sample}} < 0.9 \times \text{CO}$)

GREY ZONE Patient sample (mean) OD value from 10 % above to 10 % below CO
repeat test 2 - 4 weeks later - with new patient sample
($0.9 \times \text{CO} \leq \text{Mean } \text{OD}_{\text{sample}} \leq 1.1 \times \text{CO}$)
Results in the second test again in the grey zone \Rightarrow **NEGATIVE**

POSITIVE Patient sample (mean) OD value more than 10 % above CO
(Mean $\text{OD}_{\text{sample}} > 1.1 \times \text{CO}$)

8 QUALITY CONTROL

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results. Use controls at both normal and pathological levels.

It is also recommended to make use of national or international Quality Assessment programs in order to ensure the accuracy of the results.

If the results of the assay do not fit to the established acceptable ranges of control materials patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above mentioned items without finding any error contact your distributor or DRG directly.

9 ASSAY CHARACTERISTICS

9.1 Specificity of Antigen

The antigen used for the DRG Varicella zoster Virus IgM ELISA shows no cross-reactivity to the listed antibodies.

Sample positive to	n	expected Diagnosis	found Diagnosis
HSV-1 IgM	6	negative	negative
HSV-2 IgM	7	negative	negative
HSV 1+2 IgM	7	negative	negative
EBV IgM	8	negative	negative
CMV IgM	8	negative	negative
Rheumatoid Factor positive	39	negative	negative
samples without known antibody titers	39	negative	38 negative; 1 positive

9.2 Analytical Sensitivity

The analytical sensitivity of the DRG ELISA was calculated by adding 2 standard deviations from the mean of 20 replicate analyses of the negative control and was found to be 0.48 DU/mL ($OD_{450} = 0.022$).

9.2.1 Assay Dynamic Range

The measuring range of the assay is 0.48 DU/mL – 60 DU/mL.

9.1 Diagnostic Sensitivity and Specificity

The diagnostic specificity is defined as the probability of the assay of scoring negative in the absence of the specific analyte.

The diagnostic sensitivity is defined as the probability of the assay of scoring positive in the presence of the specific analyte.

In total, 851 samples are analyzed, including samples from 99 pregnant women, 227 children, 32 ANA-positives, 23 HAMA-positives, 116 cancer patients, and 340 apparently healthy donors, and 14 donors positive for VZV IgM.

Cut-off	N	True Positive (TP)	True Negative (TN)	False Positive (FP)	False Negative (FN)
10 DU	851	14	828	9	0

The diagnostic sensitivity is 100.0 %.

The diagnostic specificity is 98.9 %.

9.2 Method Comparison

The DRG VZV IgM ELISA was compared with the Novatec VZV IgM ELISA. 79 serum samples are assayed.

		Novatec	
		pos.	neg.
DRG ELISA	pos.	16	0
	neg.	0	63

Agreement: 100%

9.3 Repeatability and Reproducibility

9.3.1 Repeatability (within-run precision)

The within-run precision of the DRG ELISA was determined by 20 measurements of 12 samples covering the measuring range of the ELISA.

	n	Diagnosis	Mean OD	Within-run CV (%)
Sample 1	20	negative	0.203	5.5
Sample 2	20	negative	0.303	4.9
Sample 3	20	negative	0.201	6.3
Sample 4	20	low positive	0.836	6.7
Sample 5	20	low positive	0.906	3.7
Sample 6	20	low positive	0.927	3.1
Sample 7	20	positive	1.441	5.3
Sample 8	20	positive	1.256	5.5
Sample 9	20	positive	1.487	5.0
Sample 10	20	high positive	1.996	3.3
Sample 11	20	high positive	1.712	4.5
Sample 12	20	high positive	2.747	1.9
Mean Intra Assay CV (%):				4.6

9.3.2 Reproducibility (Between-run)

The between-run precision of the DRG ELISA was determined in 20 independent runs with 2 replicates per run.

Control/Sample	Mean OD	SD	Between-run CV (%)	n
Pos. Control	1.661	0.059	3.5	40
Cut-Off Control	0.602	0.032	5.3	40
Sample 1	1.273	0.046	3.6	40
Sample 2	1.379	0.054	3.9	40
Sample 3	1.468	0.061	4.1	40

10 LIMITATIONS OF USE

Bacterial contamination or repeated freeze-thaw cycles of the specimen may affect the absorbance values. In immunocompromised patients and newborns serological data only have restricted value.

10.1 Interfering Substances

10.1.1 Matrix Interference

No interference (bias $< \pm 20\%$) was found for addition of interferent up to the concentration stated in the table below.

In general, hemolytic, icteric or lipemic samples should be avoided.

Bilirubin	0.5 mg/mL
Hemoglobin	4.0 mg/mL
Triglyceride	30.0 mg/mL

11 LEGAL ASPECTS

11.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. In case of any doubt or concern please contact DRG.

11.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient. Diagnosis of an infectious disease should not be established on the basis of a single test result. A precise diagnosis should take into consideration clinical history, symptomatology as well as serological data.

Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

11.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid.

Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

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SHORT INSTRUCTIONS FOR USE

	All reagents and specimens must be allowed to come to room temperature (18-25°C) before use.
	Dispense 100 µl of Controls into appropriate wells.
	Dispense 100 µl of diluted sample into selected wells. (Please note special sample treatment, point 5.3!)
	Cover wells with foil. Incubate for 60 minutes at 37 °C.
	Briskly shake out the contents of the wells. Rinse the wells 5 times with diluted Wash Solution (300 µL per well). Strike the wells sharply on absorbent paper to remove residual droplets
	Dispense 100 µl of Enzyme-Conjugate into each well.
	Incubate for 30 minutes at room temperature.
	Briskly shake out the contents of the wells. Rinse the wells 5 times with diluted Wash Solution (300 µL per well). Strike the wells sharply on absorbent paper to remove residual droplets
	Add 100 µl of Substrate Solution to each well.
	Incubate for 15 minutes at room temperature.
	Stop the reaction by adding 100 µl of Stop Solution to each well.
	Determine the absorbance of each well at 450 nm.