## Instructions for Use

# MetCombi Plasma ELISA (Metanephrene–Normetanephrene)

IVD

CE

REF EIA-4082

Σ 2 x 96

**DRG**

DRG Instruments GmbH, Germany  
Frauenbergstraße 18, 35039 Marburg  
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50  
Website: [www.drg-diagnostics.de](http://www.drg-diagnostics.de)  
E-mail: [drg@drg-diagnostics.de](mailto:drg@drg-diagnostics.de)

**DRG**

DRG International, Inc., USA  
841 Mountain Ave., Springfield, NJ 07081  
Phone: (973) 564-7555, Fax: (973) 564-7556  
Website: [www.drg-international.com](http://www.drg-international.com)  
E-mail: [corp@drg-international.com](mailto:corp@drg-international.com)

**Please use only the valid version of the Instructions for Use provided with the kit.**  
**Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.**  
**Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.**  
**Por favor, use sólo la versión válida de las instrucciones de uso que se suministran con el kit.**  
**Utilisez seulement la version valide des Instructions d'utilisation fournies avec le kit.**  
**Por favor, usar a versão válida das instruções de utilização fornecidas com o kit.**

<b>ENGLISH .....</b>	<b>2</b>
<b>DEUTSCH.....</b>	<b>12</b>
<b>REFERENCES / LITERATURE .....</b>	<b>22</b>
<b>SYMBOLS USED.....</b>	<b>23</b>

**Related Products:**

- Metanephrite Plasma ELISA (EIA-4313)
- Normetanephrite Plasma ELISA (EIA-4319)

**1 INTRODUCTION****1.1 Intended use and principle of the test**

Enzyme Immunoassay for the quantitative determination of free metanephrite and free normetanephrite in plasma.

The determination of metanephrite and normetanephrite helps in the detection of paragangliomas and pheochromocytomas.

Metanephrite (metadrenaline) and normetanephrite (normetadrenaline) are first extracted using an ion exchange matrix followed by an acylation process.

The subsequent competitive ELISA uses the microtiter plate format. The antigen is bound to the solid phase of the microtiter plate. The acylated standards, controls and samples compete with the solid phase bound analytes for a fixed number of antibody binding sites. After the system is in equilibrium, free antigen and free antigen-antibody complexes are removed by washing. The antibody bound to the solid phase is detected by an anti-rabbit IgG-peroxidase conjugate using TMB as a substrate resulting in a colour reaction. The reaction is monitored at a wavelength of 450 nm.

Quantification of unknown samples is achieved by comparing their absorbance with a reference curve prepared with known standard concentrations. Manual processing of the ELISA is recommended. The use of automatic laboratory equipment is the responsibility of the user.

**This in-vitro-diagnostic is for professional use only.**

**1.2 Clinical application**

Metanephrite and normetanephrite are the metabolites of the catecholamines epinephrine and norepinephrine, respectively [1]. Cells derived from neuroendocrine tumors (e. g., pheochromocytoma and paraganglioma) are known to produce catecholamines, which are secreted episodically via vesicles into the blood stream [2, 3]. But beside this, a small portion of the catecholamines is metabolized inside the tumor cells to the corresponding catecholamines metabolites – namely metanephrite, normetanephrite (and 3-methoxytyramine in the case of dopamine) – which are secreted at low levels continuously into the blood stream [4, 5]. Recent studies and publications have shown that the quantification of these plasma free metanephrites and plasma free normetanephrites is the most accurate biochemical marker for the clinical diagnosis of pheochromocytoma and paraganglioma in patients [5 – 13]. Pheochromocytoma and paraganglioma are rare neuroendocrine tumors and occur with an estimated annual incidence of 1 – 8 cases per 1,000,000 [10, 14].

Therapeutic consequences should never be based on laboratory results alone, even if these results are assessed in accordance with the quality criteria of the method. Any laboratory result is only a part of the total clinical picture of the patient.

Only in cases where the laboratory results are in an acceptable agreement with the overall clinical picture of the patient, it can be used for therapeutic consequences.

**2 PROCEDURAL CAUTIONS, GUIDELINES, WARNINGS AND LIMITATIONS****2.1 Procedural cautions, guidelines and warnings**

1. This kit is intended for professional use only. Users should have a thorough understanding of this protocol for the successful use of this kit. Only the test instruction provided with the kit is valid and must be used to run the assay. Reliable performance will only be attained by strict and careful adherence to the instructions provided.
2. This assay was validated for a certain type of sample as indicated in Intended Use (please refer to Chapter 1). Any off-label use of this kit is in the responsibility of the user and the manufacturer cannot be held liable.
3. The principles of Good Laboratory Practice (GLP) must be followed.
4. In order to reduce exposure to potentially harmful substances, wear lab coats, disposable protective gloves and protective glasses where necessary.
5. If serious incidents should occur in connection with this product, they should be reported to the manufacturer and the competent national authorities.
6. All kit reagents and specimens should be brought to room temperature and mixed gently but thoroughly before use. For dilution or reconstitution purposes, use deionized, distilled, or ultra-pure water. Avoid repeated freezing and thawing of reagents and specimens.
7. The microplate contains snap-off strips. Unused wells must be stored at 2 °C - 8 °C in the sealed foil pouch with desiccant and used in the frame provided. Microtiter strips which are removed from the frame for usage should be marked accordingly to avoid any mix-up.
8. Duplicate determination of sample is highly recommended.
9. Once the test has been started, all steps should be completed without interruption. Make sure that the required reagents, materials, and devices are prepared for use at the appropriate time.
10. Incubation times do influence the results. All wells should be handled in the same order and time intervals.
11. To avoid cross-contamination of reagents, use new disposable pipette tips for dispensing each reagent, sample, standard and control.
12. A standard curve must be established for each run.
13. The controls should be included in each run and fall within established confidence limits. The confidence limits are listed in the QC-Report provided with the kit.

14. Do not mix kit components with different lot numbers within a test and do not use reagents beyond expiry date as shown on the kit labels.
15. Avoid contact with Stop Solution containing 0.25 M H<sub>2</sub>SO<sub>4</sub>. It may cause skin irritation and burns. In case of contact with eyes or skin, rinse off immediately with water.
16. TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant volume of water and skin with soap and abundant water. Rinse contaminated items before reuse.
17. For information about hazardous substances included in the kit please refer to Safety Data Sheet (SDS). The Safety Data Sheet for this product is available upon request.
18. Kit reagents must be regarded as hazardous waste and disposed of according to national regulations.
19. The expected reference values reported in this test instruction are only indicative. It is recommended that each laboratory establishes its own reference intervals.
20. In case of any severe damage to the test kit or components, the manufacturer has to be informed in writing, at the latest, one week after receiving the kit. Severely damaged single components must not be used for a test run. They must be stored properly until the manufacturer decides what to do with them. If it is decided that they are no longer suitable for measurements, they must be disposed of in accordance with national regulations.
21. The results obtained with this test kit should not be taken as the sole reason for any therapeutic consequence but have to be correlated to other diagnostic tests and clinical observations.
22. Reagents of this kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by approved procedures. All reagents however, should be treated as potential biohazards in use and for disposal.

## 2.2 Limitations

Any inappropriate handling of samples or modification of this test might influence the results.

Commercially available synthetic normetanephrine or metanephrine is always a mixture of the D- and L-form. This has important implications if synthetic metanephries are used to enrich native samples. The antibodies used in this kit have a specific D- and L-form recognition rate. Please contact the manufacturer for details in case synthetic metanephrine and normetanephrine were used to enrich native samples.

### 2.2.1 Interfering substances and proper handling of specimens

Samples containing precipitates or fibrin strands might cause inaccurate results.

**Metanephrine:** Hemolytic samples (up to 0.5 mg/mL hemoglobin), icteric samples (up to 0.5 mg/mL bilirubin) and lipemic samples (up to 17 mg/mL triglycerides) have no influence on the assay results.

**Normetanephrine:** Hemolytic samples (up to 1 mg/mL hemoglobin), icteric samples (up to 0.25 mg/mL bilirubin) and lipemic samples (up to 17 mg/mL triglycerides) have no influence on the assay results.

If the concentrations cannot be estimated and there are doubts as to whether the above limit values for hemolytic, icteric or lipemic samples are complied with, the samples should not be used in the assay.

### 2.2.2 Drug and food interferences

Medications like antihypertensive agents, antidepressants, antipsychotics, sympathomimetics and L-DOPA can influence plasma metanephries levels. Caffeinated beverages, nicotine, and mood-enhancing drugs can also affect plasma metanephries levels. In addition, stress and physical strain should be avoided shortly before sampling.

### 2.2.3 High-Dose-Hook effect

No hook effect was observed in this test.

## 3 STORAGE AND STABILITY

Store kit and reagents at 2 °C - 8 °C until expiration date.

Do not use kit and components beyond the expiry date indicated on the kit labels.

Once opened, the reagents are stable for 2 months when stored at 2 °C - 8 °C. Once the resealable pouch of the ELISA plate has been opened, care should be taken to close it tightly again including the desiccant.

## 4 MATERIALS

### 4.1 Contents of the kit

	<b>FOILS</b>	<b>Adhesive Foil</b> – ready to use
Content:	Adhesive foils in a resealable pouch	
Number:	2 x 4 foils	
	<b>WASH-CONC 50x</b>	<b>Wash Buffer Concentrate</b> – concentrated 50x
Content:	Buffer with a non-ionic detergent and physiological pH	
Volume:	2 x 20 ml/vial, purple cap	
	<b>CONJUGATE</b>	<b>Enzyme Conjugate</b> – ready to use
Content:	Goat anti-rabbit immunoglobulins conjugated with peroxidase	
Volume:	2 x 12 mL/vial, red cap	
Description:	Species is goat	
	<b>SUBSTRATE</b>	<b>Substrate</b> – ready to use
Content:	Chromogenic substrate containing 3,3',5,5'-tetramethylbenzidine, substrate buffer and hydrogen peroxide	
Volume:	2 x 12 mL/vial, black cap	
	<b>STOP-SOLN</b>	<b>Stop Solution</b> – ready to use
Content:	0.25 M sulfuric acid	
Volume:	2 x 12 mL/vial, grey cap	
	<b>U ADR MN</b>	<b>Metanephrite Microtiter Strips</b> – ready to use
Content:	1 x 96 wells (12x8) antigen precoated microwell plate in a resealable blue pouch with desiccant	
	<b>U NAD NMN</b>	<b>Normetanephrite Microtiter Strips</b> – ready to use
Content:	1 x 96 wells (12x8) antigen precoated microwell plate in a resealable yellow pouch with desiccant	
	<b>MN-AS</b>	<b>Metanephrite Antiserum</b> – ready to use
Content:	Rabbit anti-metanephrite antibody in buffer with proteins and non-mercury preservative, blue coloured	
Volume:	1 x 6 mL/vial, blue cap	
Description:	Species of antibody is rabbit, species of protein in buffer is bovine	
	<b>NMN-AS</b>	<b>Normetanephrite Antiserum</b> – ready to use
Content:	Rabbit anti-normetanephrite antibody in buffer with proteins and non-mercury preservative, yellow coloured	
Volume:	1 x 6 mL/vial, yellow cap	
Description:	Species of antibody is rabbit, species of protein in buffer is bovine	
	<b>ADJUST-BUFF</b>	<b>Adjustment Buffer</b> – ready to use
Content:	TRIS buffer	
Volume:	1 x 10 mL/vial, yellow cap	
	<b>ACYL-CONC</b>	<b>Acylation Concentrate</b> – concentrated
Content:	Acylation reagent in DMSO	
Volume:	1 x 1.5 ml/vial, white cap	
Hazard pictograms:		
Signal word:	GHS05 GHS08	
Hazardous ingredients:	Danger	
	Succinic anhydride	
Hazard statements:	H314 Causes severe skin burns and eye damage. H317 May cause an allergic skin reaction. H334 May cause allergy or asthma symptoms or breathing difficulties if inhaled.	
Precutionary statements:	P260 Do not breathe mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection. P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304+P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.	

P310 Immediately call a POISON CENTER/doctor.  
 P501 Dispose of contents/container to an authorised waste collection point.

EUH-statements: EUH071 Corrosive to the respiratory tract.

**ASSAY-BUFF****Assay Buffer – ready to use**

Content: 25% organic solvent

Volume: 1 x 30 mL/vial, orange cap

GHS02

Signal word: Warning

**EXTRACT-PLATE 96****Extraction Plate – ready to use**

Content: 1 x 96 well plate, precoated with ion-exchanger in a resealable pouch

**CLEAN-CONC 25X****Cleaning Concentrate – concentrated 25x**

Content: Buffer with sodium acetate

Volume: 1 x 20 ml/vial, brown cap

**ELUTION-BUFF****Elution Buffer – ready to use**

Content: 0.1 M sodium hydroxide, dark purple coloured

Volume: 1 x 14 ml/vial, green cap

GHS05

Signal word: Danger

Hazard statements: H314 Causes severe skin burns and eye damage.

Precautionary statements: P280 Wear protective gloves, protective clothing, eye protection.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a doctor, a POISON CENTER.

P501 Dispose of contents/container to an authorised waste collection point.

**EQUA-REAG****Equalizing Reagent – ready to use**

Content: Human serum, negative for HIV I/II, HBsAg and HCV

Volume: 1 x 14 ml/vial, white cap

Description: Species is human

## 4.2 Calibration and Controls

### Standards and Controls – ready to use

Component	Colour/ Cap	Concentration [pg/mL]		Concentration [pmol/L]		Volume/ Vial
		MN	NMN	MN	NMN	
<b>STANDARD A</b>	white	0	0	0	0	4 mL
<b>STANDARD B</b>	yellow	36	72	183	393	4 mL
<b>STANDARD C</b>	orange	120	240	608	1,310	4 mL
<b>STANDARD D</b>	blue	360	720	1,825	3,931	4 mL
<b>STANDARD E</b>	grey	1,200	2,400	6,084	13,104	4 mL
<b>STANDARD F</b>	black	3,600	7,200	18,252	39,312	4 mL
<b>CONTROL 1</b>	green	Refer to QC-Report for expected value and acceptable range!				4 mL
<b>CONTROL 2</b>	red					4 mL

Conversion: metanephrite [pg/mL] x 5.07 = metanephrite [pmol/L]

normetanephrite [pg/mL] x 5.46 = normetanephrite [pmol/L]

Content: Acidic buffer with non-mercury stabilizer, spiked with a defined quantity of metanephrite and normetanephrite.

## 4.3 Additional materials required but not provided in the kit

- Water (deionized, distilled, or ultra-pure)
- Absorbent material (paper towel)

## 4.4 Additional equipment required but not provided in the kit

- Calibrated precision pipettes to dispense volumes between 20 µL – 350 µL; 3 mL
- Microtiter plate washing device (manual, semi-automated or automated)
- ELISA reader capable of reading absorbance at 450 nm and if possible 620 – 650 nm
- Microtiter plate shaker (shaking amplitude 3 mm; approx. 600 rpm)
- Vortex mixer

## 5 SAMPLE COLLECTION HANDLING AND STORAGE

### EDTA- or Heparin-Plasma

Whole blood should be collected into centrifuge tubes (Monovette or Vacutte) containing EDTA or heparin as anti-coagulant and centrifuged (according to manufacturer's instructions) immediately after collection. When in doubt, it is recommended that hemolytic, icteric, and lipemic samples not be used in the assay (see 2.2.1).

Storage: up to 3 days at 2 °C - 8 °C,  
for longer period (up to 6 months) at -20 °C.

Repeated freezing and thawing should be avoided.

## 6 TEST PROCEDURE

Allow all reagents and samples to reach room temperature and mix thoroughly by gentle inversion before use. Number the Extraction Plate and microwell plates (Microtiter Strips which are removed from the frame for usage should be marked accordingly to avoid any mix-up). Duplicate determinations are recommended.

The binding of the antisera and of the enzyme conjugate and the activity of the enzyme are temperature dependent. The higher the temperature, the higher the absorption values will be. Varying incubation times will have similar influences on the absorbance. The optimal temperature during the enzyme immunoassay is between 20 °C - 25 °C.

If the product is prepared in parts, unused wells in Extraction Plates should be covered to avoid contamination. After preparation, the used wells must be labelled to prevent double use.

During the overnight incubation at 2 °C - 8 °C with the antiserum, the temperature should be uniform all over the ELISA plate to avoid any drift and edge-effect.

**⚠** The use of a microtiter plate shaker with the following specifications is mandatory: shaking amplitude 3 mm; approx. 600 rpm. Shaking with differing settings might influence the results.

### 6.1 Preparation of reagents and further notes

#### Wash Buffer

Dilute the 20 mL Wash Buffer Concentrate **WASH-CONC 50X** with water to a final volume of 1000 mL.

Storage: 2 months at 2 °C - 8 °C

**Cleaning Buffer**

Dilute the 20 mL Cleaning Concentrate **CLEAN-CONC 25X** with water to a final volume of 500 mL.

Storage: 2 months at 2 °C - 8 °C

**Acylation Solution**

**⚠** As the Acylation Solution is only **stable for a maximum of 3 minutes**, it should not be prepared before starting the assay. Therefore, its preparation is described in the protocol in chapter 6.3, step 3 and chapter 6.4, step 3. Discard after use!

**Metanephrite and Normetanephrite Microtiter Strips**

In rare cases residues of the blocking and stabilizing reagent can be seen in the wells as small, white dots or lines. These residues do not influence the quality of the product.

**Extraction Plate**

In rare cases residues of the cation exchanger can be seen in the wells as small, black dots or lines. These residues do not influence the quality of the product.

**6.2 Preparation of samples – Extraction**

**⚠** The extraction procedure is the same for metanephrite and normetanephrite and has to be done only once.

The following extraction procedure can be run with 200 µL or 250 µL of plasma sample.

The procedure for 250 µL plasma is highlighted in grey and italicised and may be used in case higher supernatant volumes for pipetting to the subsequent ELISA are preferred.

The ELISA procedure itself is not affected by this alternative protocol.

1. Pipette **20 µL** of **standards** and **controls** into the respective wells of the **EXTRACT-PLATE 96**.  
**Alternatively pipette 25 µL of standards and controls.**
2. Add **20 µL** **STANDARD A** to all wells intended for the **plasma samples**.  
**Alternatively add 25 µL STANDARD A.**
3. Add **200 µL** of **EQUA-REAG** to the wells with **standards and controls**.  
**Alternatively add 250 µL of EQUA-REAG.**
4. Pipette **200 µL** of **plasma samples** to the respective wells.  
**Alternatively pipette 250 µL of plasma samples.**
5. Incubate plate **for 2 h at RT** (20 °C - 25 °C) on a **shaker** (approx. 600 rpm).
6. Empty plate and blot dry by tapping the inverted plate on absorbent material.
7. Pipette **250 µL** of **ASSAY-BUFF** into all wells. Incubate the plate for **5 min at RT** (20 °C - 25 °C) on a **shaker** (approx. 600 rpm). Empty plate and blot dry by tapping the inverted plate on absorbent material.
8. Wash the plate **3 times** by adding **350 µL** of **Cleaning Buffer**, **discarding** the content and **blotting dry each time** by tapping the inverted plate on absorbent material.
9. Pipette **100 µL** of **ELUTION-BUFF** into all wells.  
**Alternatively pipette 125 µL of ELUTION-BUFF.**  
*Please note: The colour changes caused by the elution buffer can vary between standards and samples.*
10. Cover plate with **FOILS**. Incubate **15 min at RT** (20 °C - 25 °C) on a **shaker** (approx. 600 rpm). Remove the **FOILS**.

**⚠ Do not decant the supernatant thereafter!**

The following volumes of the supernatant are needed for the subsequent ELISA:

**Metanephrite 50 µL**

**Normetanephrite 25 µL**

### 6.3 Metanephrine ELISA

1. Pipette **25 µL** of **ADJUST-BUFF** into all wells of the **Metanephrine Microtiter Strips** **W ADR MN**.
2. Pipette **50 µL** of the extracted **standards, controls and samples** into the respective wells.  
*Please hold the Extraction Plate at a slight angle in order to facilitate this pipetting step.*
3. Preparation of **Acylation Solution**:  
Pipette **80 µL ACYL-CONC** to **3 mL water** and mix thoroughly.
4. Pipette **25 µL** of the freshly prepared **Acylation Solution** into all wells.
5. Incubate for **15 min** at **RT (20 °C - 25 °C)** on a **shaker** (approx. 600 rpm).
6. Pipette **50 µL** of the **Metanephrine Antiserum** **MN-AS** into all wells.
7. Cover the plate with **FOILS**, shake for **1 min** at **RT (20 °C - 25 °C)** on a **shaker** and incubate for **15 h – 20 h** (overnight) at **2 °C - 8 °C** without shaking.
8. Remove the foil. Discard or aspirate the contents of the wells. Wash the plate **4 times** by adding **300 µL** of **Wash Buffer, discarding** the content and **blotting dry each time** by tapping the inverted plate on absorbent material.
9. Pipette **100 µL** of the **CONJUGATE** into all wells.
10. Incubate for **30 min** at **RT (20 – 25 °C)** on a **shaker** (approx. 600 rpm).
11. Discard or aspirate the contents of the wells. Wash the plate **4 times** by adding **300 µL** of **Wash Buffer, discarding** the content and **blotting dry each time** by tapping the inverted plate on absorbent material.
12. Pipette **100 µL** of the **SUBSTRATE** into all wells and incubate for **20 – 30 min** at **RT (20 – 25 °C)** on a **shaker** (approx. 600 rpm). **Avoid exposure to direct sunlight!**
13. Add **100 µL** of the **STOP-SOLN** to all wells and shake the microtiter plate shortly.
14. **Read** the absorbance of the solution in the wells within 10 min, using a microplate reader set to **450 nm** (if available a reference wavelength between 620 nm and 650 nm is recommended).

### 6.4 Normetanephrine ELISA

1. Pipette **25 µL** of **ADJUST-BUFF** into all wells of the **Normetanephrine Microtiter Strips** **W NAD NMN**.
2. Pipette **25 µL** of the extracted **standards, controls and samples** into the respective wells.  
*Please hold the Extraction Plate at a slight angle in order to facilitate this pipetting step.*
3. Preparation of **Acylation Solution**:  
Pipette **80 µL ACYL-CONC** to **3 mL water** and mix thoroughly
4. Pipette **25 µL** of the freshly prepared **Acylation Solution** into all wells.
5. Incubate for **15 min** at **RT (20 °C - 25 °C)** on a **shaker** (approx. 600 rpm).
6. Pipette **50 µL** of the **Normetanephrine Antiserum** **NMN-AS** into all wells.
7. Cover the plate with **FOILS**, shake for **1 min** at **RT (20 °C - 25 °C)** on a **shaker** and incubate for **15 h – 20 h** (overnight) at **2 °C - 8 °C** without shaking.
8. Remove the foil. Discard or aspirate the contents of the wells. Wash the plate **4 times** by adding **300 µL** of **Wash Buffer, discarding** the content and **blotting dry each time** by tapping the inverted plate on absorbent material.
9. Pipette **100 µL** of the **CONJUGATE** into all wells.
10. Incubate for **30 min** at **RT (20 °C - 25 °C)** on a **shaker** (approx. 600 rpm).
11. Discard or aspirate the contents of the wells. Wash the plate **4 times** by adding **300 µL** of **Wash Buffer, discarding** the content and **blotting dry each time** by tapping the inverted plate on absorbent material.
12. Pipette **100 µL** of the **SUBSTRATE** into all wells and incubate for **20 min – 30 min** at **RT (20 °C - 25 °C)** on a **shaker** (approx. 600 rpm). **Avoid exposure to direct sunlight!**
13. Add **100 µL** of the **STOP-SOLN** to all wells and shake the microtiter plate shortly.
14. **Read** the absorbance of the solution in the wells within 10 min, using a microplate reader set to **450 nm** (if available a reference wavelength between 620 nm and 650 nm is recommended).

## 7 CALCULATION OF RESULTS

Measuring range	Metanephrine	Normetanephrine
	15.1 – 3,600 pg/mL	22.8 – 7,200 pg/mL

The standard curve, which can be used to determine the concentration of the unknown samples, is obtained by plotting the absorbance readings (calculate the mean absorbance) of the standards (linear, y-axis) against the corresponding standard concentrations (logarithmic, x-axis) using a concentration of 0.001 pg/mL for Standard A (this alignment is mandatory because of the logarithmic presentation of the data).

Use non-linear regression for curve fitting (e.g. 4-parameter, marquardt).

**⚠ This assay is a competitive assay. This means: the OD-values are decreasing with increasing concentrations of the analyte. OD-values found below the standard curve correspond to high concentrations of the analyte in the sample and have to be reported as being positive.**

The concentrations of the **samples and controls** can be read directly from the standard curve.

Samples found with concentrations higher than the highest standard (Standard F) should be diluted accordingly with the included Equalizing Reagent **EQUA-REAG** and must be re-assayed.

### Conversion:

metanephrine [pg/mL] x 5.07 = metanephrine [pmol/L]

normetanephrine [pg/mL] x 5.46 = normetanephrine [pmol/L]

### 7.1 Expected reference value

It is strongly recommended that each laboratory should determine its own reference values.

The expected reference values indicated below are based on method comparison studies to LC-MS/MS [2] with blood samples taken in the sitting position.

	Metanephrine	Normetanephrine
expected reference value (ULN)	< 100 pg/mL	< 216 pg/mL
typical pathological range	up to 3,534 pg/mL	up to 8,500 pg/mL


For the interpretation of the results, a grey area has to be considered. This grey area does not depend on the methodology used and is reflected in a slight to mediate increase in metanephrine and normetanephrine up to 4 times the upper cut-off [15].

Approx. 20% of the tumors are found in this grey area, especially in the case of the Hereditary Syndrome, incidental tumors and in sporadic cases of pheochromocytomas with a diameter less than 1 cm.

In case of a result in the grey area, it is recommended to collect a new sample together with an anamnesis concerning especially influences like the medication and age of the patient. If the sample continues to be found in the grey area, a clonidine suppression test may be considered [16, 17].

### 7.2 Typical standard curve

**⚠ Examples, Do not use for calculation!**

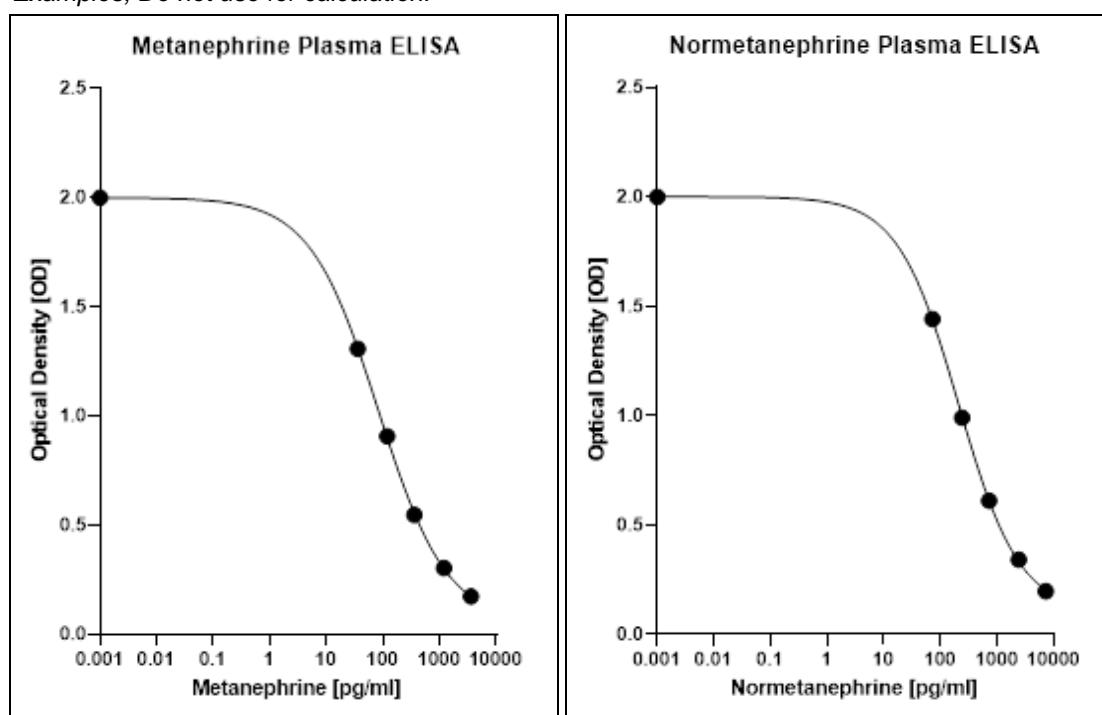## 8 CONTROL SAMPLES

It is recommended to use control samples according to national regulations. Use controls at both normal and pathological levels. Commercially obtained control samples should be treated like unknown samples. Control samples should fall within established confidence limits (please refer to "Limitations" chapter 2.2). The confidence limits of the kit controls are indicated on the QC-Report.

## 9 ASSAY CHARACTERISTICS

### 9.1 Performance data

Analytical Sensitivity		
	Metanephine	Normetanephine
Limit of Blank (LOB)	9.9 pg/mL	11.7 pg/mL
Limit of Detection (LOD)	14.9 pg/mL	17.9 pg/mL
Limit of Quantification (LOQ)	15.1 pg/mL	22.8 pg/mL

Substance	Cross Reactivity [%]	
	Metanephine	Normetanephine
Metanephine	100	0.72
Normetanephine	0.05	100
3-Methoxytyramin	< 0.01	6.5*
Adrenaline	< 0.01	< 0.01
Noradrenaline	< 0.01	< 0.01
Dopamin	< 0.01	< 0.01
Vanillic mandelic acid	< 0.01	< 0.01
Homovanillic acid	< 0.01	< 0.01
L-DOPA	< 0.01	< 0.01
L-Tyrosin	< 0.01	< 0.01
Tyramine	< 0.01	< 0.01
Acetaminophen	< 0.01	< 0.01

\*Normetanephine concentrations are not influenced by 3-methoxytyramine in case of normal 3-methoxytyramine concentrations. Only very high 3-methoxytyramine concentrations found in rare cases of exclusively dopamine secreting tumors can cause false positive results.

Precision							
Intra-Assay				Inter-Assay			
	Sample	Mean [pg/mL]	CV [%]		Sample	Mean [pg/mL]	CV [%]
Metanephine	1	66.3	11.4	Metanephine	1	67.8	17.6
	2	122	13.5		2	134	12.7
	3	308	10.6		3	319	11.0
	4	783	9.2		4	847	7.5
Normetanephine	1	149	9.5	Normetanephine	1	156	10.6
	2	282	9.1		2	287	5.0
	3	734	8.2		3	769	5.1
	4	1,956	10.5		4	1,949	5.9

Lot-to-Lot				
	Sample	Mean ± SD [pg/mL]	CV [%]	
Metanephine (n=6)	1	97.7 ± 16.5	16.9	
	2	870 ± 117	13.5	
Normetanephine (n=6)	1	231 ± 29.9	13.0	
	2	1,688 ± 116	6.9	

Recovery was determined according to the CLSI standard EP 34 1st ed.

<b>Recovery</b>			
	Range [pg/mL]	Mean [%]	Range [%]
Metanephrene	20.9 – 1,291	102	90 – 109
Normetanephrene	63.6 – 2,004	90	84 – 93

<b>Linearity</b>			
	Serial dilution up to	Mean [%]	Range [%]
Metanephrene	1:64	107	101 – 124
Normetanephrene	1:64	98	92 – 102

<b>Method Comparison: ELISA vs. LC-MS/MS [13]</b>			
Metanephrene	$y = 0.91x + 1.8; r^2 = 0.96; n = 46$		
Normetanephrene	$y = 0.93x + 13; r^2 = 0.99; n = 48$		

<b>Diagnostic Performance [2]</b>				
	Diagnostic Specificity [%]	Diagnostic Sensitivity [%]	Positive Predictive Value (PPV) [%]	Negative Predictive Value (NPV) [%]
Metanephrene and Normetanephrene	96	100	89	100
	Positive Likelihood Ratio (LR+)			Negative Likelihood Ratio(LR-)
	25			0

## 9.2 Metrological Traceability

The values assigned to the standards and controls of the MetCombi Plasma ELISA are traceable to SI Units by weighing with quality-controlled analyte.

<b>Standards and Controls</b>	
	Uncertainty [%]
Metanephrene	2.0
Normetanephrene	2.0

<b>MetCombi Plasma ELISA</b>	
	Expanded Uncertainty [%] k = 2*
Metanephrene	25.7
Normetanephrene	10.8

\*This defines an interval about the measured result that will include the true value with a probability of 95%.

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- Metanephrine Plasma ELISA (EIA-4313)
- Normetanephrine Plasma ELISA (EIA-4319)

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## 10 REFERENCES/LITERATURE / REFERENZEN/LITERATUR

1. Lee, S.M., et al., Development and validation of liquid chromatography-tandem mass spectrometry method for quantification of plasma metanephries for differential diagnosis of adrenal incidentaloma. Ann Lab Med, 2015. 35(5): p. 519-22.
2. Peaston, R.T., et al., Performance of plasma free metanephries measured by liquid chromatography-tandem mass spectrometry in the diagnosis of pheochromocytoma. Clin Chim Acta, 2010. 411(7-8): p. 546-52.
3. de Jong, W.H., et al., Dietary influences on plasma and urinary metanephries: implications for diagnosis of catecholamine-producing tumors. J Clin Endocrinol Metab, 2009. 94(8): p. 2841-9.
4. Boot, C., et al., Single-centre study of the diagnostic performance of plasma metanephries with seated sampling for the diagnosis of phaeochromocytoma/paraganglioma. Ann Clin Biochem, 2017. 54(1): p. 143-148.
5. Shen, Y., et al., A simple and robust liquid chromatography tandem mass spectrometry assay for determination of plasma free metanephries and its application to routine clinical testing for diagnosis of pheochromocytoma. Biomed Chromatogr, 2019. 33(10): p. e4622.
6. Anas, S.S. and S.D. Vasikaran, An audit of management of patients with borderline increased plasma-free metanephries. Ann Clin Biochem, 2010. 47(Pt 6): p. 554-8.
7. Eisenhofer, G., et al., Plasma metanephries in renal failure. Kidney Int, 2005. 67(2): p. 668-77.
8. Eisenhofer, G., et al., Pheochromocytoma catecholamine phenotypes and prediction of tumor size and location by use of plasma free metanephries. Clin Chem, 2005. 51(4): p. 735-44.
9. Eisenhofer, G., et al., Biochemical Diagnosis of Chromaffin Cell Tumors in Patients at High and Low Risk of Disease: Plasma versus Urinary Free or Deconjugated O-Methylated Catecholamine Metabolites. Clin Chem, 2018. 64(11): p. 1646-1656.
10. Grouzmann, E., et al., Diagnostic accuracy of free and total metanephries in plasma and fractionated metanephries in urine of patients with pheochromocytoma. Eur J Endocrinol, 2010. 162(5): p. 951-60.
11. Pussard, E., A. Chaouch, and T. Said, Radioimmunoassay of free plasma metanephries for the diagnosis of catecholamine-producing tumors. Clin Chem Lab Med, 2014. 52(3): p. 437-44.
12. Unger, N., et al., The value of immunoassays for metanephries in the biochemical diagnosis of pheochromocytomas. Horm Metab Res, 2009. 41(9): p. 676-9.
13. de Jong, W.H., et al., Plasma free metanephrine measurement using automated online solid-phase extraction HPLC tandem mass spectrometry. Clin Chem, 2007. 53(9): p. 1684-93.
14. Mullins, F., et al., Enzyme-linked immunoassay for plasma-free metanephries in the biochemical diagnosis of phaeochromocytoma in adults is not ideal. Clin Chem Lab Med, 2011. 50(1): p. 105-10.
15. Eisenhofer, G., et al., Biochemical diagnosis of pheochromocytoma: how to distinguish true- from false-positive test results. J Clin Endocrinol Metab, 2003. 88(6): p. 2656-66.
16. Remde, H., et al., Improved Diagnostic Accuracy of Clonidine Suppression Testing Using an Age-Related Cutoff for Plasma Normetanephrine. Hypertension, 2022. 79(6): p. 1257-1264.
17. Bartsch, D.K. and K. Lorenz. S2k-Leitlinie - Operative Therapie von Nebennierentumoren 2017; Available from: [https://www.awmf.org/uploads/tx\\_szleitlinien/088-008I\\_S2k\\_Operative-Therapie\\_Nebennierentumoren\\_2019-07.pdf](https://www.awmf.org/uploads/tx_szleitlinien/088-008I_S2k_Operative-Therapie_Nebennierentumoren_2019-07.pdf).

For updated literature or any other information please contact your local supplier.

Once EUDAMED is fully in place, the Summary of Safety and Performance can be found here:  
<https://ec.europa.eu/tools/eudamed>.

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